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Vitrebond™ Light Cure glass ionomer liner/base, manufactured by 3M™ ESPE™, is comprised of a powder and a liquid component. The powder component is a light sensitive fluoro-aluminosilicate glass. The liquid component is a light sensitive polymeric acid. The composition is a true glass ionomer exhibiting the major characteristics of glass ionomer products – it bonds to both structure, releases fluoride, and is a biocompatible material. Additionally, Vitreond liner/base offers the unique combination of a prolonged working time with a very short set time achieved by exposure to light from a dental visible light curing unit. Polymerization by light exposure not only eliminates the set time waiting period known to auto set, it also provides the same handling and physical properties. Vitreond light cure glass ionomer liner/base is indicated for lining and basing applications under composite, amalgam, ceramic, and metal restorations.

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This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as described in MSDSs can be obtained from [www.3MSEPE.com](#) or contact your local distributor.

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as described in MSDSs can be obtained from [www.3MSEPE.com](#) or contact your local distributor.

PRECAUTIONARY INFORMATION FOR PATIENTS:
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylic allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.
PRECAUTIONARY INFORMATION FOR DENTAL PERSONNEL:
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as described in MSDSs can be obtained from [www.3MSEPE.com](#) or contact your local distributor.

INSTRUCTIONS FOR USE:
1. Pulp Protection: Vitrebond liner/base is not indicated for direct pulp capping. If a pulp exposure occurs, cover the exposure with a calcium hydroxide material. Place the Vitrebond liner/base over the calcium hydroxide and surrounding dentin to seal and protect the exposure.
2. Dentin Pretreatment: Dentin pretreatment is not recommended. The use of smear layer cleansers such as polyactic acid based solutions results in decreased adhesion of the liner/base.
3. Dispensing:
3.1. Both the Vitrebond light cure glass ionomer liner/base powder and the liquid are light sensitive. To prevent deactivation by light exposure, protect both from ambient light **by replacing jar and vial caps immediately** after dispensing and dispense powder and liquid just prior to mixing and placement.

- Mixing:** Use a small cement spatula to rapidly mix (10–15 seconds) all the powder into the liquid. The mixed cement should have a smooth consistency and glossy appearance. Excessive spatulation will shorten working time. To minimize water evaporation and maximize working time, confine spatulation of the powder and liquid to a small area of the mixing pad, about one inch (2.5 cm) diameter.
- Application and Curing:** Avoid water and saliva contamination during application and setting of the liner/base. Rubber dam is the best means of isolation. Apply the mixed liner/base to the dentin surface of the prepared cavity in a thin layer (1/2mm or less) using a ball applicator. The Vitrebond liner/base has a minimum working time of 2 minutes/40 seconds at room temperature. Higher temperatures will shorten working time. Cure the Vitrebond liner/base by exposing it for 30 seconds to light from a 3M ESPE light curing unit or other curing unit of comparable intensity. The delayed auto-setting mechanism of the Vitrebond liner/base will ensure an eventual cure of material shielded from light polymerization such as in undercut areas. Where thicker applications of the liner/base are desired, best adhesion can be achieved by first placing and light curing a thin layer followed by placement of a second layer up to 2mm in depth and light curing for 30 seconds.

- Adhesive system:** Continue with the bonding step of the restorative procedure starting with etching. Etching on the liner/base is not deleterious. Refer to specific product instructions for details of use.
- Storage AND USE:**
This product is designed to be stored and used at room temperature. Shelf life at room temperature is 36 months. Ambient temperatures routinely higher than 27°C/80°F, or lower than 10°C/50°F may reduce shelf life. See outer package for expiration date.
Both the Vitrebond liner/base powder and liquid are light sensitive. To prevent deactivation of the materials, protect from ambient light. **Replace jar and vial caps immediately** after dispensing.

- Different the scope and bottles using an intermediate level disinfection process (liquid contact as recommended by the CDC Centers for Disease Control and Prevention and endorsed by the ADA (American Dental Association), Guidelines for Infection Control in Dental Health-Care Settings –2003 (Vol. 52, No. RR- 17), Center for Disease Control and Prevention.
- Do not expose materials to elevated temperature or intense light.
Disposal – See the Material Safety Data Sheet (available at [www.3MSEPE.com](#) or through your local subsidiary) for disposal information.

- Customer Information:** No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

CAUTION: US Federal Law restricts this device to sale or use on the order of a dental professional.

WARRANTY:
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. Use is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

LIMITATION OF LIABILITY:
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether arising under contract, tort, or otherwise, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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INSTRUCTIONS FOR USE:
1. Pulp Protection: Vitrebond liner/base is not intended for direct pulp capping. If a pulp exposure occurs, cover the exposure with a calcium hydroxide material. Place the Vitrebond liner/base over the calcium hydroxide and surrounding dentin to seal and protect the exposure.
2. Dentin Pretreatment: Dentin pretreatment is not recommended. The use of smear layer cleansers such as polyactic acid based solutions results in decreased adhesion of the liner/base.
3. Dispensing:
3.1. Both the Vitrebond light cure glass ionomer liner/base powder and the liquid are light sensitive. To prevent deactivation by light exposure, protect both from ambient light **by replacing jar and vial caps immediately** after dispensing and dispense powder and liquid just prior to mixing and placement.

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- Adhesive system:** Continue with the bonding step of the restorative procedure starting with etching. Etching on the liner/base is not deleterious. Refer to specific product instructions for details of use.
- Storage AND USE:**
This product is designed to be stored and used at room temperature. Shelf life at room temperature is 36 months. Ambient temperatures routinely higher than 27°C/80°F, or lower than 10°C/50°F may reduce shelf life. See outer package for expiration date.
Both the Vitrebond liner/base powder and liquid are light sensitive. To prevent deactivation of the materials, protect both from ambient light. **Replace jar and vial caps immediately** after dispensing and dispense powder and liquid just prior to mixing and placement.

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