ENGLISH
<b>General:</b> <p>3M ESPE Clinpro™ Prophy Paste contains a unique blend of polishing and cleaning agents designed for use by dental professionals in the course of standard dental hygiene procedures. The paste contains sodium fluoride at a concentration of 1.23% fluoride ion.</p> <b>Indication:</b> <p>Indicated for use in cleaning and polishing procedures, as part of a professionally administrated prophylaxis treatment.</p> <b>Precautions:</b> <ul style="list-style-type: none"><li>Clinpro Prophy Paste is to be used only by individuals professionally trained to perform dental prophylaxis.</li> <li>Excessive ingestion of fluoride can cause nausea. Clinpro Prophy Paste should be expectorated and NOT SWALLOWED.</li></ul> <b>Instructions for Use:</b> <ul style="list-style-type: none"><li>Insert unit-dose cup into supplied prophylaxis ring.</li> <li>One unit-dose cup provides enough prophylaxis paste for a single prophylaxis procedure.</li> <li>Grit cup colors and uses are defined below. <ul style="list-style-type: none"><li>Gold: Fine Grit - Gentle polishing and cleansing</li> <li>Blue: Medium Grit - Normal polishing, cleansing and plaque removal</li> <li>Red: Coarse Grit - Medium to heavy stain and plaque removal</li></ul></li></ul> <b>Storage and Use:</b> <p>Store at normal room temperature.</p> <p>No person is authorized to provide any information which deviates from the information provided in this instruction sheet.</p> <b>Warranty:</b> <p>3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.</p> <b>Limitation of Liability:</b> <p>Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.</p>

<span>44-0007-4702-0-A</span>	
<span>Dental Products</span>	
<b>3M ESPE</b>	
9/08	
<span>Naudojimo instrukcijos</span>	<span>Instrukcija de utilizare</span>
<span>Letošnias instrukcija</span>	<span>Instukcija uzycia</span>
<span>Kasutisjshlend</span>	<span>Használati utasítás</span>
<span>Kilaima Tallmatlan</span>	<span>Upute za uporabu</span>
<span>Návod k pouziti</span>	<span>Инструкция за употреба</span>
<span>Navodla za uporabo</span>	<span>Инструкция по применению</span>
<span>Návod na použitie</span>	<span>Instrucions for use</span>

<span>Prophylaxis Polishing Paste With Fluoride</span>	<span>Profylaktická pasta na leštenie s fluoridom</span>
<span>Профилактическая, полирующая паста со фтором</span>	<span>Profilaktična pasta s fluorom za poliranje zob</span>
<span>Профилактична полираща паста с флуор</span>	<span>Profylaktická lešticí pasta s fluoridem</span>
<span>Preventivna pasta za poliranje sa fluoridom</span>	<span>Fluorürü Profilaksi Cilalama Pati</span>
<span>Profilaktikus polírozó paszta fluoriddal</span>	<span>Profülaktiline poleerimispaste fluoriidiga</span>
<span>Profilaktyczna pasta do polerowania, z fluorem</span>	<span>Profilakses un pulišanas pasta ar fluoridu</span>
<span>Pastă de lustruire pentru profilaxie cu fluorură</span>	<span>Profilaktinë poliruojamoji pasta su fluoridais</span>

<span>Profilaktikus polírozó paszta fluoriddal</span>	<span>Profülaktiline poleerimispaste fluoriidiga</span>
<span>Profilaktyczna pasta do polerowania, z fluorem</span>	<span>Profilakses un pulišanas pasta ar fluoridu</span>
<span>Pastă de lustruire pentru profilaxie cu fluorură</span>	<span>Profilaktinë poliruojamoji pasta su fluoridais</span>

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3M, ESPE and Clinpro are trademarks of 3M or 3M ESPE AG. © 3M 2008. All rights reserved. 44-0007-4702-0-A