Detaseal® bite

Bite registration material, silicone based, addition curing fast setting – high final hardness

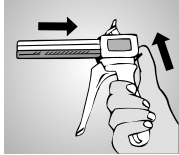
Fig. 1

1. Mixing and Dosing

Extrude material from the cartridge with Automix gun (Fig. 1). Place cartridge into mixing gun and remove closure cap by rotation. Eject a small amount of material (bleeding) until equal flow is being extruded from both orifices (Fig. 2). Insert mixing cannula into guide grooves on cartridge and cannula and rotate clockwise to a stop (Fig. 3). Store cartridge with mixing cannula attached for subsequent use.

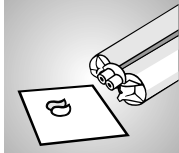
Fig. 2

2. Instruction for use

Extrude **Detaseal® bite** from cartridge with mixing cannula or with attached contouring tip onto teeth (Fig. 4). At terminal occlusion application from vestibular is possible. Let registration cure in terminal occlusion and remove from the mouth after 1 min. 30 sec.

Fig. 3

3. Disinfection

After removal from the mouth rinse registrate under lukewarm water. Disinfection for 15 min. in 2 % glutaraldehyde.

Fig. 4


Important hints

- Remove any retraction astringents or mouth wash solutions prior to the registration – could affect setting reaction.
- Do not intermix with condensation curing silicones.
- Avoid staining of clothing. Cured impression materials cannot be removed.
- Remove all residues of registration material from the patient's mouth.

Further information:

Silicone based materials are proven a million times. On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to get tested the allergenic potential before the application of the material.

For use by trained specialists.

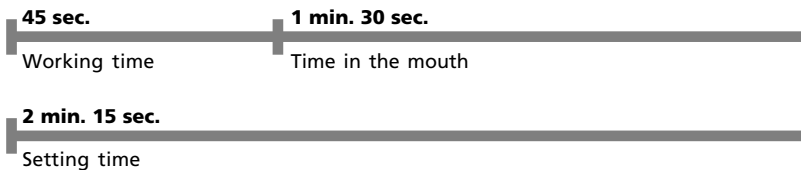Indications for use:

- Bite registration with crown and bridge impressions and for orthodontic indications, key for intraoral registration

Technical Data:

DIN 13903

- **Mixed volume:** 50 ml (cartridge)
- **Mixing ratio:** 1 : 1
- **Colour code:** base: yellow catalyst: white
- **Mixing time:** omitted (Automix-system)
- **Working time:** approx. 45 sec.*
- **Setting time:** approx. 2 min. 15 sec.*
- **Time in mouth:** approx. 1 min. 30 sec.
- **Shore hardness A:** approx. 90
- **Linear dimensional change:** < 0.2 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

Ordering information:

Detaseal® bite
Standard packing **02431**
2 x 50 ml cartridges
12 mixing cannulas, 6 mm
12 contouring tips

Mixing cannulas **02344**
6 mm, 48 pcs.

Detaseal® bite Automix2
Standard packing **02139**
2 x 50 ml cartridges
12 mixing cannulas, rose
12 contouring tips

Ecopacking **02020**
8 x 50 ml cartridges
48 mixing cannulas, rose
48 contouring tips

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