smartseal® & loc

GB
Instructions for use

transparent / natural opaque F

Fissure sealant, light-curing, composite-based, hydrophilic, natural opaque with fluoride

Fig. 1

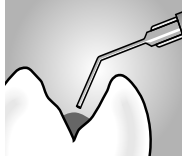
Fig. 2

Fig. 3

Fig. 4

Fig. 5

Applications

smartseal® & loc is used for prophylactic sealing of fissures, pits, carious lesions in deciduous teeth and permanent teeth. It can also be used for sealing after a finished orthodontic treatment; sealing of damaged enamel surfaces.

1. Etching

Clean the enamel surface (Fig. 1) and remove all traces of cleaning paste. Delineate the area of treatment if necessary and dry with oil-free air. Apply the etching gel, e.g. **smartseal® etch**, only to the tooth enamel (Fig. 2). Allow the etching gel to act for 20 - 30 sec. Then rinse off the etching gel for approx. 20 sec. with running water (Fig. 3). Remove excess water with an air stream (oil-free) or absorb with a cotton swab. **The enamel surface should not be dried completely but kept moist.** Avoid contact between the etched surface and saliva.

2. Syringe application

Remove the cap from the application syringe. Fit black application needles onto the application syringe and lock in place. The needle may be bent slightly. Hold the application syringe vertically, vent the cannula channel by pressing the plunger carefully, and fill with material up to the application syringe. The sealant flows uniformly when slight to medium pressure is applied. **smartseal® & loc** should preferably be applied to the moist enamel surface but it can also be applied to enamel which has started to dry. Introduce **smartseal® & loc** to the fissure thinly (Fig. 4) and allow it to penetrate for about 20 sec. Before light curing check for any bubbles and remove them with a probe or cannula.

3. Curing

Using a halogen or LED lamp, cure each surface for approx. 20 seconds (Fig. 5). Then check the sealant and margins with a probe. Supplement any omitted areas and remove any excess carefully, e.g. using a finishing diamond bur. If necessary, polish the sealant with rubber polisher.

Important working hints

- If the etched surface is contaminated, e.g. with saliva, etching must be repeated.
- Only expose to light from an operating lamp briefly because this causes the start of polymerisation and viscosity increases.
- Avoid contact with mucous membranes, eyes and skin. In the event of accidental contact, rinse with plenty of running water, and consult an eye specialist if necessary.
- Material which has not cured completely can cause irritation and sensitization if it comes into contact with skin or mucous membranes.
- Close carefully immediately after use. Remove the cannula and use the original cap. Use a new application needle the next time. Store away from light.

For use by trained specialists.

Technical data:

DIN EN ISO 6874 - class 2

- **Volume:** 1.5 g syringe
- **Dispensing an applying:** Direct application by application syringe and application needles or application brush
- **Product colour:** smartseal® & loc: transparent
smartseal® & loc F: natural opaque
- **Curing:** Halogen light > 500 mW/cm²: approx. 20 sec. LED > 300 mW/cm²: approx. 20 sec.
- **Application:** Store at ambient temperature
- **Storage:**

Ordering information:

smartseal® & loc

Combi pack 02612
1 x 1,5 g smartseal® & loc application syringe
1 x 1,5 g smartseal® & loc F application syringe
12 application needles, black, long

smartseal® & loc transparent

Standard packing 02573
2 x 1,5 g smartseal® & loc application syringe
12 application needles, black, long

smartseal® & loc F natural opaque

Standard packing 02581
2 x 1,5 g smartseal® & loc F application syringe
12 application needles, black, long

smartseal® etch

Standard packing 02326
1 x 2 ml smartseal® etch application syringe
10 application needles, blue

Application needles 02456
25 needles, black, long

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