ENGLISH

General Information
3M™ ESPE™ Sof-Lex™ Spiral Finishing and Polishing Wheels are a 2-step, single use finishing and polishing system. The Sof-Lex Spiral Wheels use the same mandrel as 3M ESPE™ Sof-Lex™ Finishing and Polishing Discs. Having a single shape and using the Sof-Lex™ Mandrel eliminates the task of changing out multi-shaped instruments typically needed to polish restorations. The Sof-Lex Spiral Wheels are made of a thermoplastic elastomer impregnated with aluminum oxide particles.

Indications
The beige Sof-Lex Spiral Finishing Wheel is indicated to smooth and remove scratches in restorations developed during contouring. The white Sof-Lex Spiral Polishing Wheel is indicated for the final high-gloss polishing. Water spray is not needed during use. Sof-Lex Spiral Wheels can be used to polish surfaces of:

- Direct:**
- Composite restorations
- Resin modified glass ionomers
- Bisacrylic temporary materials

Indirect:

- Composite
- Lava™ Ultimate CAD/CAM Restorative

- Precious and semi-precious metal

Precautionary Information for Dental Personnel
Wear eye protection when using this product.

DO NOT use with high speed hand pieces.

Operating hand piece at speeds greater than 25,000 rpm may cause Sof-Lex Spiral Wheels and mandrel to separate or Sof-Lex Spiral Wheels to fragment resulting in injury.

Excessive, heavy pressure (over 100 grams force) may cause Sof-Lex Spiral Wheels to separate or fragment resulting in injury.

Using Sof-Lex instruments while the hub (center of wheel) is in contact with teeth (especially occlusal or incisal edge) may result in Sof-Lex instruments separating from mandrel resulting in injury.

Sof-Lex instruments are designed to be used **only** with Sof-Lex mandrels. Some resistance should be felt when placing a Sof-Lex Spiral Wheel on a mandrel. If it appears loose, replace instrument.

Heads of Sof-Lex mandrels will wear over time. Replace mandrel if Sof-Lex instruments persistently feel loose.

Do not attempt to sterilize or reuse Sof-Lex instruments. Sterilization can adversely affect their performance.

3M ESPE MSDS information can be obtained from www.3MESPE.com or contact your local subsidiary.

Instructions for Use

Gross reduction: Use Sof-Lex coarse discs or trimming and finishing diamonds or carbide burs to remove excess (or air inhibited layer) restorative material.

Final contour: Complete and refine final restoration contours and margins with Sof-Lex medium discs, fine or very fine diamond or finishing burs. Restoration should have its desired form and a smooth, clean surface.

Smooth surface (scratch removal) with beige Sof-Lex Spiral Finishing Wheel:

- Insert a Sof-Lex mandrel into a low speed hand piece.
- Attach the beige Sof-Lex Spiral Finishing Wheel by pushing it onto the Sof-Lex mandrel.
- Optimum results are obtained when the Sof-Lex system is used on dry surfaces.

4. A forward handpiece direction is recommended for operation of the Sof-Lex Spiral Wheels. Running the Sof-Lex Spiral Wheels in reverse direction can alter the shape of the instruments significantly.

5. For best results, use light (30 grams force bends the bristle tips) to moderate pressure (60 grams force causes a significant bend in the bristles) with hand piece speed of 10,000-20,000 rpm and keep the instrument in motion over the restoration surface.

- 3.5.1. If the hub of the wheel is being deformed (bent), too much pressure is being applied.

6. All surfaces of the instrument (top, bottom, edge) can be utilized for maximum access to the restorative surface.

High gloss polish with white Sof-Lex Spiral Polishing Wheel:

- When satisfied with results of the beige Sof-Lex Spiral Finishing Wheel, remove wheel by grasping edge and pulling up and off the mandrel.
- Rinse debris from restoration and tooth surfaces.
- For a higher gloss place the white Sof-Lex Spiral Polishing Wheel onto the mandrel and repeat steps 3.3 to 3.6.
- After bringing surface to high gloss shine rinse with water.

5. No other additional steps are necessary.

6. *The Sof-Lex Spiral Finishing and Polishing Wheels are single use only! Discard after use.*

Storage and Use

Sof-Lex instruments are single use items.

- Do not attempt to sterilize the Sof-Lex instruments.
- Properly dispose of used Sof-Lex instruments in accordance with your healthcare facility's policy.

Sof-Lex Mandrel re-use:

Sof-Lex Mandrels must be cleaned and high level disinfected or sterilized between patients. See Sof-Lex Mandrel Instructions for Use for further detail.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Caution: U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the products for user’s application. If this product is defective within the warranty period, user’s exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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