smartfluorid® lacquer

GB
Instructions for use

Fluoride lacquer for long-term fluoridation and remineralisation, fast drying

Fig. 1

Fig. 2

Fig. 3

Fig. 4

Range of indications

Intensive caries prophylaxis, desensitising oversensitive cervical areas and hypersensitive dentine by fluoridation. Forms a protective film against thermal and mechanical effects.

1. Preparation

Clean the surfaces to be treated (we recommend pretreat with **smartprotect**® Desensitizer) and dry. Clean and dry working areas. Use cotton wool rolls to keep the working area dry. Apply **smartprotect**® with a **smartbrush** or application brush and massage in for 10 sec. (Fig. 1). Dry completely in the air current (Fig. 2).

2. Application

Shake **smartfluorid**® lacquer well before use. The level of the lacquer and a metal ball facilitate mixing the lacquer in the bottle. Put a drop of **smartfluorid**® lacquer onto a flexible brush and apply with light touches to the surfaces (Fig. 3).

3. Drying

Allow the lacquer to take effect for 10 - 20 seconds and dry completely with the air jet until a polymer film forms (Fig. 4). Repeat the treatment two or three times at intervals of a few days with hypersensitive cervical areas and crown margins or similar indications. Dry areas already treated and apply another layer of **smartfluorid**® lacquer.

Important working hints

- Allow each application of **smartfluorid**® lacquer to take effect on the treated surface for as long as possible.
- It is essential to instruct the patient not to clean the tooth surface for 12 hours following application.
- The consistency of **smartfluorid**® lacquer should be thin enough after shaking that it flows easily from the drop dispenser. Thin with a few drops of 2-propanol if necessary.
- When used correctly, the lacquer matrix does not dissolve until it has been completely softened and after it has released the maximum amount of fluoride.
- Applying **smartprotect**® Desensitizer beforehand has an added cleansing and desensitising effect.

For use by trained specialist staff.

Technical data:

- **Volume:**
10 ml bottle
- **Colour:**
clear
- **Drying time:**
approx. 30 - 60 sec.

Properties:

- Organic fluorides help to build up a fluoride depot for long-term fluoride release by amino fluoride and calcium fluoride.
- Forms a protective film that insulates the tooth structure mechanically and thermally.

Composition:

Fluoride lacquer in 2-propanol

- **Relevant ingredients:**
Olaflur and calcium fluoride.
- **Storage:**

Ordering information:

smartfluorid® lacquer

Standard pack 02216

1 x 10 ml
smartfluorid® lacquer
20 flexible brushes, blue

Flexible brushes 02396

100 brushes, blue

smartprotect®

Standard pack 02159

1 x 7 ml bottle
20 x smartbrush
1 flowchart

smartprotect® soft

Standard pack 02582

1 x 5 ml bottle
5 x 0.5 ml single container, reclosable
5 smartbrush, white
1 flowchart with 5 patient information booklets

Single Pack 02583

10 x 0.5 ml single container, reclosable
2 flowcharts with 5 patient information booklets

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